## Gentle Hair Removal Results





Male Back | 3 months post treatment | J. Waibel, M.D.






Female Underarm | Post treatment | M. Ercan, M.D.





Beard Bumps | Post treatment | O. Zerpa, M.D.

of patients report after four to six treatments.

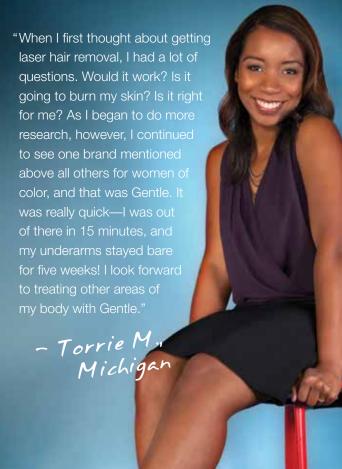

#### Science, Results, Trust.

Syneron and Candela combine the latest science with a level of expertise that ensures impressive results. Our customer understanding, superior to that of any company in our industry, has earned us the trust of physicians and patients worldwide. With breakthrough technologies currently in the pipeline, we are poised to maintain our global leadership long into the future.





© 2014. All rights reserved. Syneron and the Syneron logo are registered trademarks of Syneron Medical, Ltd. Candela and the Candela logo are registered trademarks of Candela Corporation, Gentle Hair Removal, DCD and Dynamic Cooling Device are trademarks of Candela Corporation and may be registered in certain jurisdictions. 01974 V1 0114 PB75911EN-NA



www.GetGentleHairRemoval.com

# Laser Hair Removal Comfortable Enough to Be Called Gentle

No matter what your skin type, there's a Gentle Hair Removal treatment that's right for you. Only Gentle systems feature the patented Dynamic Cooling Device,<sup>™</sup> and super-fast treatment times make Gentle the most comfortable laser hair removal available!

#### What can I expect during laser treatment?


A handpiece delivers a laser pulse into the targeted areas on your skin, and a controlled amount of therapeutic heat safely and effectively damages the hair follicles. Treatments take anywhere from a few minutes to about an hour. depending on the area being treated.

#### Do Gentle Hair Removal treatments hurt?

Gentle systems feature a patented Dynamic Cooling Device (DCD), which minimizes discomfort by releasing a quick burst of liquid cryogen before and after each laser pulse. This cryogen quickly evaporates, cooling the outer layers of the skin.

Some patients report a sensation similar to being snapped with a rubber band, but that generally resolves quickly

> with DCD cooling. A topical anesthetic may be used prior to treatment.

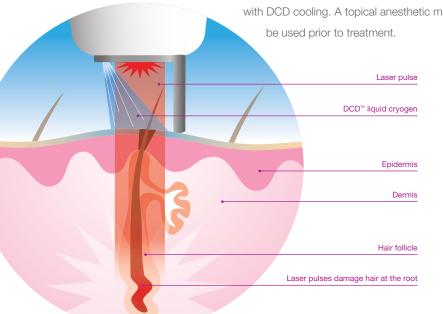

### Is Gentle Hair Removal safe for darker skin tones?


There's a safe, effective and comfortable Gentle treatment option for every skin tone. Consult with your physician about the best treatment for your individual needs.



Gentle Hair Removal is safe and effective for all skin types!

#### How many treatment sessions will I need?

You'll need a few treatments to get optimal results. That's because hair grows in three stages:

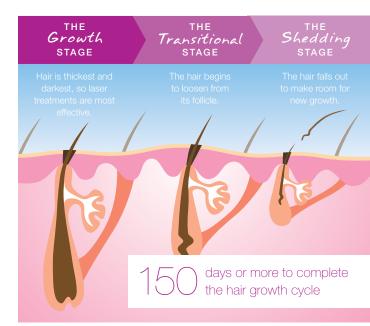

The hairs on your body are not always at the same stage, so you'll need to undergo a few treatments, an average of six weeks apart, to catch all of them in the growth stage.









